# medmaxx

Shield yourself.

 **optrel**<sup>®</sup>  
swiss made 

ASSEMBLY INSTRUCTIONS	3
HEADBAND	7
SPARE PARTS	8
ENGLISH	10
FRANÇAIS	11
DEUTSCH	12
SVENSKA	13
ITALIANO	14
ESPAÑOL	15
PORTUGUÊS	16
NEDERLANDS	17
SUOMI	18
DANSK	19
NORSK	20
POLSKI	21
ČEŠTINA	22
РУССКИЙ	23
中文	24
MAGYAR	25
TÜRKÇE	26
日本語	27
ΕΛΛΗΝΙΚΑ	28
БЪЛГАРСКИ	29
SLOVENSKÝ	30
SLOVENSKI	31
ROMÂNĂ	32
EESTI	33
LIETUVIŲ	34
LATVIEŠU	35
MALTI	36
GAEILGE	37
HRVATSKI	38


# Face shield

## Assembly instruction medmaxx


instruction video : scan QR-Code or following link:  
<https://medical.optrel.com/medmaxx/>



- 1** Peel off the protection foil on both sides as shown (inside and outside) - only the edges, not the whole foil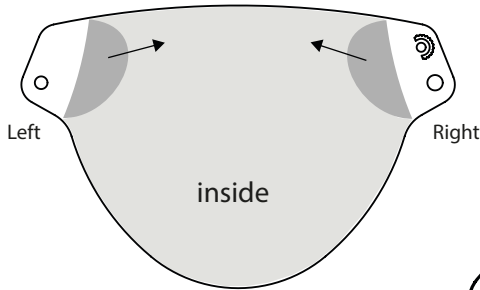


What you need:

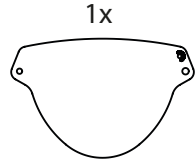
- 2** ! Please make sure that the bolt fits into the cutout section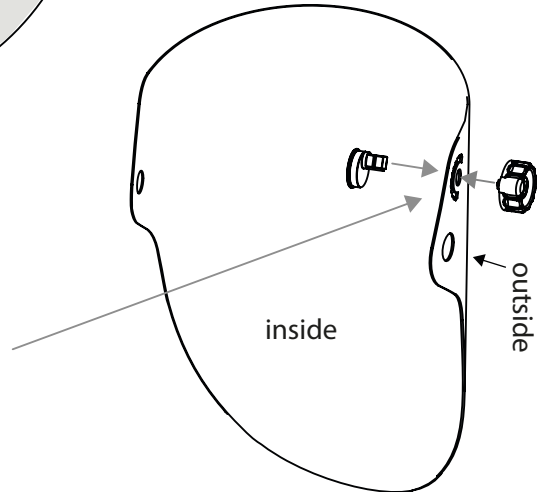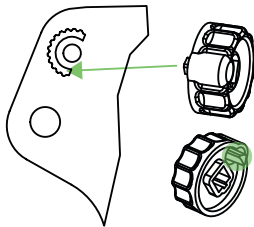

- 3** ! Press together, use your thenar to get the needed pressure or an item to get a targeted hit.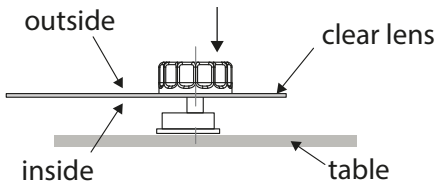

- 4** ! As soon as you hear the click, this section is finished.

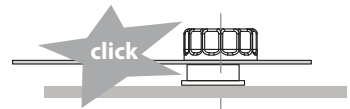
! Make sure that the Excenter bolt is flush with the excenter knob (as shown in the views).


# Hints

## Hints before we start

Correct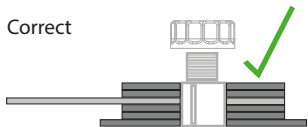


Wrong

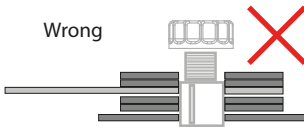! Make sure that none of the parts is in the threaded area, and all plates are guided by the notch.


! Use a pencil

There is a hint how you get the plates rightly assembled (fitted on the bolt with the notch and not in the threaded area), you only need a pencil - use it as shown in the drawing below (green marked space on the headband).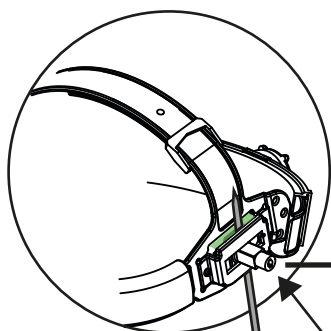


Pencil


Move the bolt to the middle position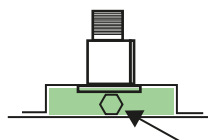

Put a pencil in between the gap to ensure that the bolt will stay up to assemble your plates.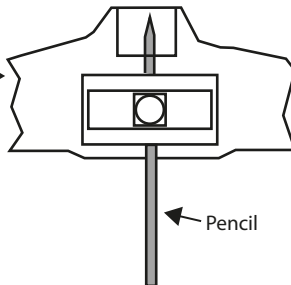



! In case of an accidental disassemble of all parts, use the sequential arrangement below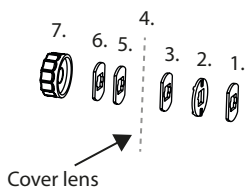

Cover lens

Left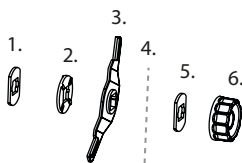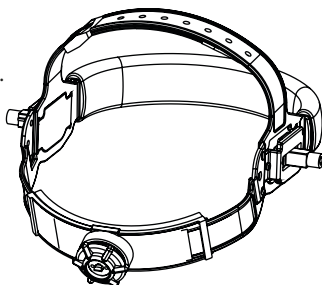

Cover lens

Right


# Right side

What you need:


1x

1x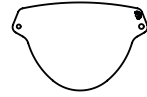

- 5** Assembly of the right side  
First of all, insert the pencil as shown. Then only remove the screw nut and the plate from the headband, place the cover lens in between, hold the headband and the cover lens in one hand and if possible stabilize the other side of the cover lens on a desk / table. Assemble the plate and the screw nut.

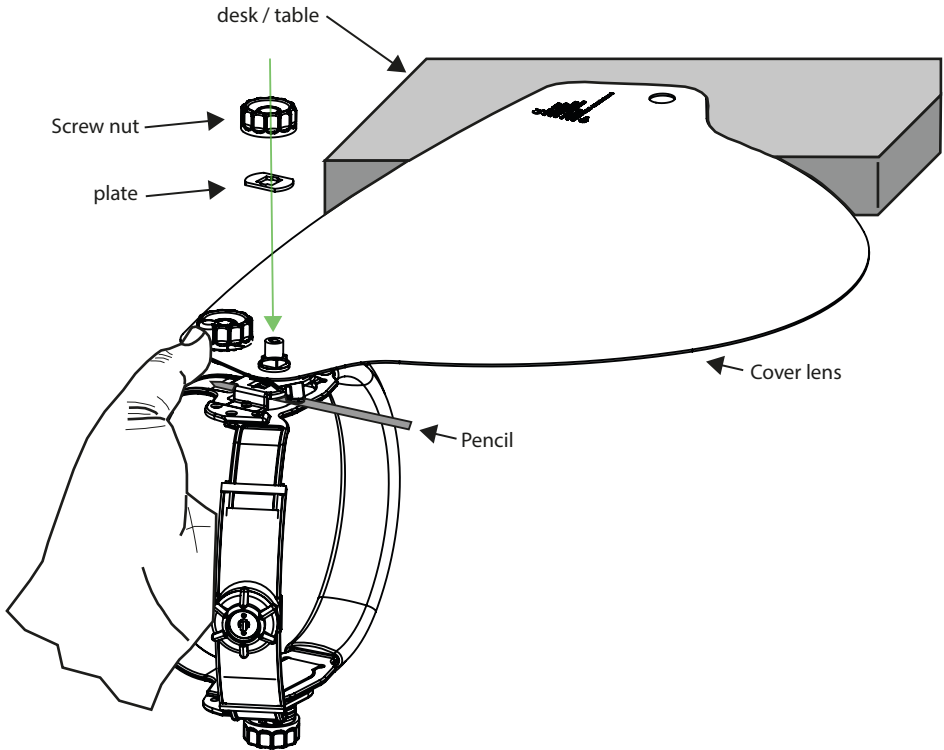
**!** Remember

Correct

Wrong

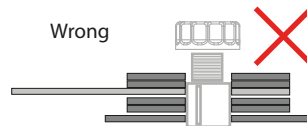
# Left side

What you need:

1x

- 6** Assembly of the left side:  
First of all, insert the pencil again on the left side. Place the headband on a desk / table and only remove the screw nut and the plate (as shown). Hold the headband and the cover lens with one hand and assemble the plates and screw nut as shown (use the additional plate from the separate bag).

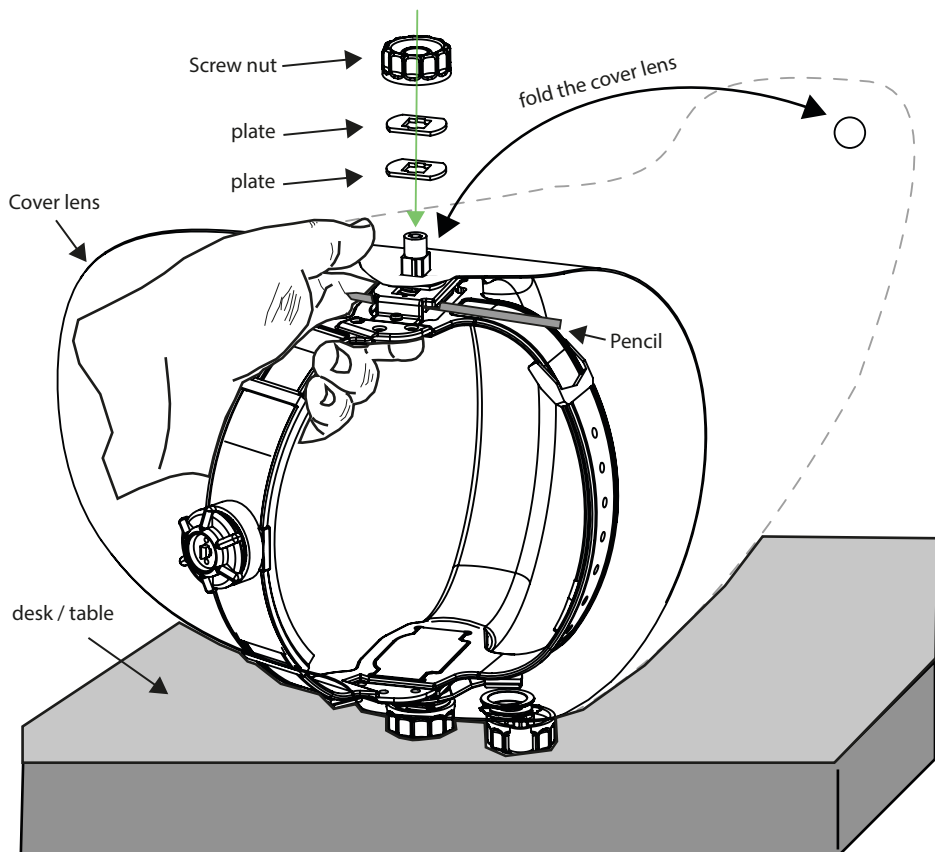
! Remember

Correct


Wrong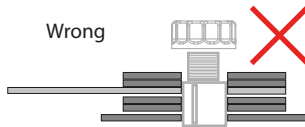


## headband settings


get the headband perfectly fitted on your head.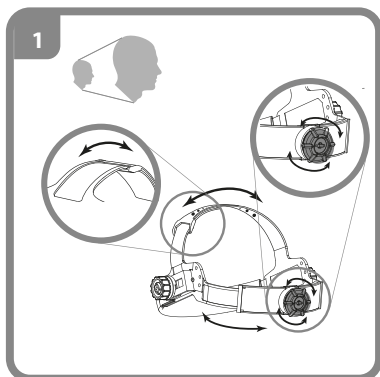


Headband: Adjust the upper adjustment band to your head size. Press the ratchet button at the back and turn it until the headband fits snugly but without pressure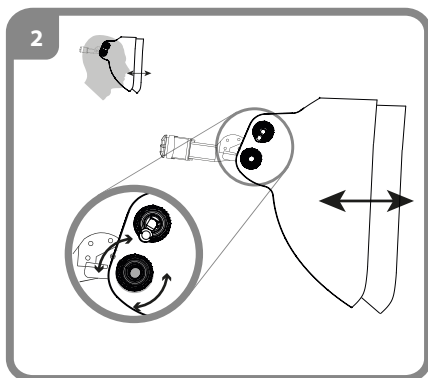

Face distance: Loosen the locking knobs to adjust the distance between the face shield and the face. Adjust both sides equally and do not tilt them. Then tighten the locking knobs again.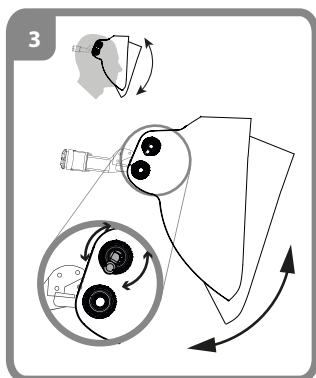

Helmet inclination: can be adjusted by turning the eccentric knob.

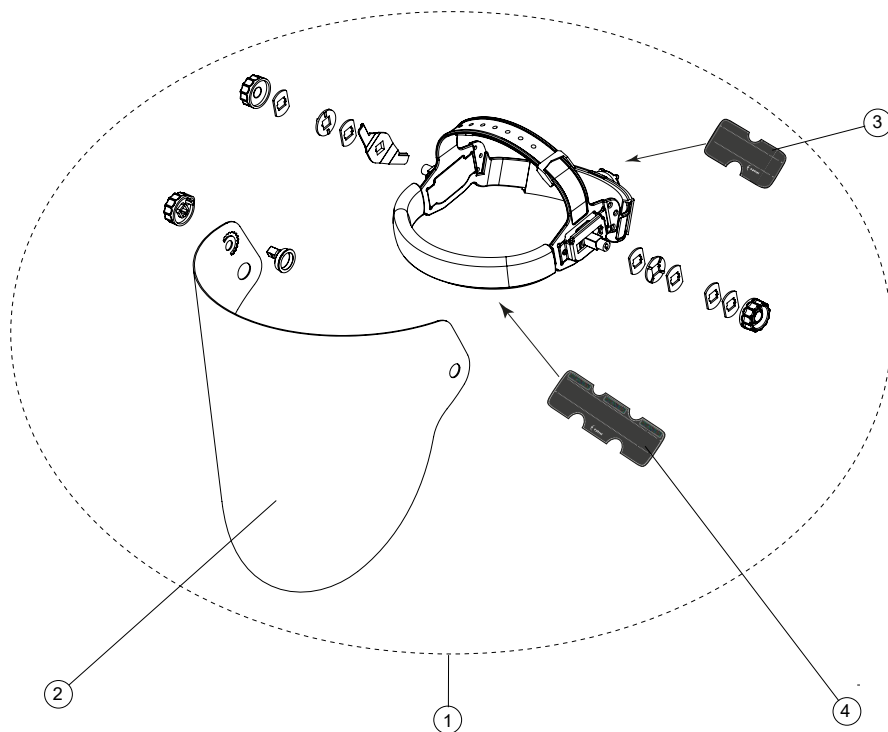<b>Optrel medmaxx</b>		
	Designation	Part. N°
1	medmaxx (complete, for self-assembly)	1100.200
2	Clear cover lens (wear part)	5000.280 (10pc)
3	Comfort band (wear part)	5004.020
4	Sweatband (wear part)	5004.073

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EN166  
AS/NZS 1337  
ANSI OS U5 Z87  
ANSI Z87  
OS 1 FT EN 166 CE

complies with  
CSA Z.94.3

The marking on the madmaxx indicates:

Marking safety protection lens:

OS	1	FT	EN	166	CE
Manufacturer		Optical class	Medium energy impact at extreme temperatures		Number of the standard

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# English

## Introduction

This product is eye and face protection with a clear view and is intended for use by medical professionals or emergency services personnel.

## Safety instructions

Read these operating instructions carefully before using the equipment. Incorrect use could endanger life and limb and void warranty claims against the manufacturer.

## Performance of the PPE / Applications / Risks

The product offers an unlimited, clear view of your patient. If the air humidity / temperature difference is high, the visor can fog up, resulting in poor visibility. If the headband is set incorrectly, there is a risk that the product will detach from the head and fall.

## Precautions and protective restrictions

- Before using the product, always check that the equipment is complete and that no damage has occurred during transport. Never use faulty or damaged equipment.
- The user must be familiar with the correct use of his/her personal protective equipment.
- Appropriate protective clothing must also be worn to protect the rest of your body. Materials in contact with the skin of the product can cause allergic reactions in sensitive individuals.
- There is a risk of bumping due to the larger size (head with visor).
- If the headband is set incorrectly, there is a risk that the product will detach from the head and fall.
- The product is intended for use by medical professionals or emergency services personnel.
- The product protects against drops and splashes of liquids. This reduces the risk of droplet infection. Due to its design, full protection against viruses, bacteria or other biological agents is only given to a limited extent.
- The product can be combined with face masks.

## Warranty & liability

For the warranty conditions, please refer to the instructions provided by your national sales organisation. For more information, please contact your retailer. The warranty only applies to material and manufacturing defects. In the event of damage caused by unintended use, tampering or any use not intended by the manufacturer, the warranty or liability shall be void. Similarly, liability and warranty are void if spare parts other than those sold by the manufacturer are used.

## Estimated service life

The product does not have an expiration date. The product can be used as long as no visible or invisible damage or malfunctions occur.

## How to use (Quick Start Guide) p. 3

1. Head strap. Set the upper adjusting strap to suit your head size. Push in the locking knob and turn it until the head strap is fitting securely but without applying pressure.
2. Distance from eyes. The distance between the protective glass and the eyes can be adjusted by releasing the locking knobs. Adjust both sides equally and do not tilt. Then tighten the locking knobs again.
3. The visor angle can be adjusted using the rotary knob.
4. After use, the product must be cleaned and disinfected thoroughly.

## Cleaning, disinfection and Maintenance

Manual cleaning: remove sweatband and comfort band. Use a clean cloth dampened with soapy water to clean the visor inside and out and allow it to dry in a clean environment.

Machine cleaning: remove sweatband and comfort band. Select a cleaning program with a maximum temperature of 70 °C.

Manual disinfection: by wiping or spraying with a commercially available disinfectant.

Machine disinfection: Select a disinfection program with a maximum temperature of 70 °C.

UV disinfection is possible.

Maintenance: The visor is to be replaced if it is

- deformed or cloudy due to high temperature
- has been scratched or deformed by a shock or mechanical influence
- became cloudy due to chemical influence

## Storage

The product must be cleaned and disinfected thoroughly before storage. Storage at room temperature and low humidity.

## Removing / installing the visor (p. 2)

To remove the visor:

- Unscrew the knobs
- Remove spacers
- Take off four

Attaching the visor: in reverse order.

## Technical data

(We reserve the right to make technical changes at any time)

Weight	250g / 8.82 oz	
Material	Headband: Visor:	Polyethylene Polycarbonate
Operating temperature	- 10°C to + 55°C / 14°F to 131°F	
Storage temperature	- 20°C to + 80°C / - 4°F to 176°F	
Shape stability	Headband: Visor:	up to 220°C / 428°F up to 137°C / 279°F
Approvals	EN166 AS/NZS 1337 ANSI OS U5 Z87 ANSI Z87 OS 1 FT EN 166 CE complies with CSA Z.94.3	

## Declaration of conformity

See the internet address on the last page.

## Legal information

This document complies with the requirements of EU Regulation 2016/425 section 1.4 of Annex II.

## Notified body

For detailed information see last page.

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ECS GmbH